Glucose blood draws made quicker and easier



Wish you could draw just a single drop of blood for glucose tests? Now you can – with the SugarPlum glucose lancet.

- The same lancet style used for adult glucose tests but sized for babies
- Two controlled depths:
 - 1.0mm depth for micropreemies to full-term babies
 - 1.8mm depth for full-term babies and adults
- Sterile, single-use needle retracts and locks after use

SugarPlum[™] Glucose Lancet

SugarPlum and NeatNick[®] – perfect companions for all your babies' blood draws!

Use SugarPlum for glucose tests that require just a drop of blood. Use our NeatNick sweeping blade lancet for PKU's, CBC's, and other tests. With NeatNick and SugarPlum you can choose the right lancet for all your babies' blood draws.

SugarPlum's simple design makes glucose draws quicker, easier and more comfortable for your babies. Press against the heel – SugarPlum automatically fires at the correct pressure. Before and after firing, the needle is safely concealed in the plastic body.

Using SugarPlum is safe and simple!

2







1 Remove protective cap.

Place the raised red platform end of lancet on the site

3 Gently press down to activate the lancet

NeatNick heel lancets



Preemie lime green



Full-term mango orange

Ordering Information

SugarPlum glucose lancet and NeatNick sweeping action heel lancet are sterile, single-use disposable "sharps" used to puncture an infant's heel for collecting a blood sample and are DEHP-free, BPA-free, and not made with natural rubber latex. See intructions for use.

SugarPlum Glucose Lancet

Item	Quantity	Part#
For micropreemies to full-term babies		
1.0 mm depth (g21) Yellow	100/box	1040043
For full-term babies and adults		
1.8 mm depth (g21) Gray	100/box	1040041

NeatNick Sweeping Action Heel Lancet

Item	Quantity	Part#
For babies weighing less than 1,800 - 2,000 g Preemie size: Lime green	100/box	1030083
For babies weighing more than 1,800 - 2,000 g Full-term size: Mango orange	100/box	1030085